
DATA SHEET



S.I.P.[®]- GOLD
Surface Improvement Pads
Includes product refs 4018, 4026 and 4028
and such other shapes as may from time to
time be made available.

Picture above: S.I.P GOLD 4026

Explanation:	Super Hydrophobic Coating on top of lenses will cause a very poor adhesion for edging pads.
Product Description:	Surface improvement Pads GOLD are designed to adhere to lenses with a Super Hydrophobic Coating. This kind of lenses are specified as Super Hydrophobic or Oleophobic lenses.
Directions for use:	SIP GOLD has a special engineered adhesive that attaches to the latest generation Super Hydrophobic coatings. It is provided with a conformable filmic backing that forms perfectly to the convex of the lens. Use your regular edging pad on top of this SIP GOLD PAD This Pad

Physical Properties:

Adhesive:	Special Engineered LSE
Backing:	Conformable yellow filmic (single sided) with yellow finger lift
Total Thickness:	0,09 mm
Liner:	Transparent PET
Shelf Life:	12 months from despatch when stored in original bags and cartons at 21°C and under 50% relative humidity

The data and details in this sheet were correct and up-to-date at the time of printing and are intended to provide information on our products and their possible applications. This sheet is not a specification and does not assure specific product characteristics or make reference to the suitability of the products for a definite application. Because Permanento cannot anticipate or control every application, we strongly recommend testing of this product under individual application conditions. In accordance with our policy of continuous improvement specification may vary.

Permanento Edge-it B.V.
Bliet 10
Industrial Estate Dombosch 1
4941 SG Raamsdonksveer
The Netherlands
Tel: +31 162 58 94 00
Fax: +31 162 52 02 14
r.hertzberger@permanento.com
w.kraaij@permanento.com